How to use your new Rehband Support - 402436, 402336

UK

Function

Provides good protection against falls and blows to the elbow. Prevention of injuries is helped through increased blood flow.

Indications

Soft tissue overload, general soreness, general impact prevention.

Application

- 1) Place the support on the elbow.
- 2) Position protective pelotte on the tip of the elbow.

Important Information

- 1) If you have any questions regarding application, please contact your local dealer or the prescriber.
- 2) The product is not flameproof, avoid temperature above 120°C/248°F.
- 3) Minor changes are allowed, providing the changes are made through the prescriber* or local dealer.
- 4) The function of the product must not be changed or compromised, the product should only be used for its original, intended indication.
- 5) SBR/neoprene products, time of use should not exceed 3-4 hours. Products made from other materials, time of use should not exceed 8 hours, unless it is recommended by the prescriber.
- 6) If the product contains natural rubber latex and you are allergic to natural rubber latex, we advise that you consult a specialist before using the support.
- 7) If you are allergic to heat, we advise you to use a product that does not contain SBR/neoprene.

* Medical- products, the prescriber must be a qualified doctor, physiotherapist, occupational therapist or orthopaedic engineer.

Washing Instructions

- 1) Mesh laundry bag is recommended.
- 2) Wash at 40 °C with similar colours.
- 3) Stretch product while wet and let dry.

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