# How to use your new Rehband Insole 206806-01

\*different colors, sizes, thicknesses, sides

	Euro	UK	US men	US women	Mondo-point
Size 1	37-38	4,5-5	5-6,5	6-7,5	23,5-24
Size 2	39-40	6-7,5	7-7,5	8-9,5	25-25,5
Size 3	41-42	8-9,5	7,9,5	10-11,5	26,5-27
Size 4	43-44	10-11,5	10-11,5	12-13,5	28-28,5
Size 5	45-46	12-12,5	12-13	14-15,5	29,5-30,5

### EN

**Function and indications: Pronation/Supination Wedge 2068\*** (\*-different colors, sizes, variants) is used to relieve pain in the foot, knee and back that are related to pronation or supination of the foot. It neutralizes the biomechanical changes in the foot, limbs and back that are associated with a pronated or supinated foot stance. When the product is placed on the medial side in the shoe to correct pronation, on lateral side supination.

**Application:** The product can be worn under both edges of the foot with the textile side up depending on the clinical diagnosis.

1)Remove the plastic layer;
2)Place on lateral side of the shoe to compensate supination and medial side to compensate pronation.

**Precautions:**

- Medical conditions that could affect product use should first be discussed with a doctor or physician.
- Use the product only for its intended purpose without modifying it.
- Application related questions should be addressed to the dealer.
- In case of problems related to the product contact the prescriber.
- Use the product in flat shoes and also it is recommended to wear socks.
- Do not continue using a visually damaged product.
- To avoid abrasion remove any sand/gravel from the shoe.
- The product is not flameproof. Avoid heat that exceeds 120°C/248°F.
- Product estimated life span is up to 6 months of everyday use.
- Each product should only be worn by one user.

**Maintenance:** 30°C hand wash. Use water or washing gel to reactivate adhesiveness. No washing powder. Store in a cool and dry place. Keep from direct sunlight.

**Materials used:** Polyurethane gel, 100% polyester.

**Disposal:** Dispose in accordance with national regulations.

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